laBeaut<sup>e</sup>

LA BEAUTE SRL  
Salerno - ITALY  
Sede Commerciale - Milano - ITALY  
[www.labeaute.it](http://www.labeaute.it)

# laBeaut<sup>e</sup>

## RF COMB

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HOT + COLD  
COMBINATION  
TO SHAPE YOUR  
SILHOUETTE »

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### AN INNOVATIVE TECHNOLOGY

With this technology, the beauty treatments lead to acting on different levels connected and interacting:

- Reconstruction of regular microcirculatory system**, with optimization of tissue nutrition and removal of metabolic waste and toxins;
- Pro-collagen effect**, with restoration of the tissue architecture and elasticity recovery;
- Control of localized fat**, with increase of the tissue metabolic activity and shapes modelling;

This technology is based on the **alternate and sequential application heating and cooling on deep dermal and hypodermal level**.

The heating is generated by bipolar resistive radiofrequency, applied by **special pregelled single-client electrodes**, specifically studied for this application. The cooling is generated by exclusive Peltier cell thermoelectrical systems.

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**RF COMB treatment plates.**  
[patented]

The 6 plates are engineered exclusively for the RF COMB method. Freeing, controlling and modulating both hypothermia and hyperthermia requires an advanced and smart system. The cooling is generated by Peltier cell thermoelectrical systems and requires exclusive materials and innovative design: **this makes the plates as a unique and unmistakable object**. The radiofrequency hyperthermia passes through special pregelled single-client electrodes, specifically studied for this application. The electrodes have a specific shape that **optimizes the treatment performances** and allows a continuous monitoring of the temperature exactly in the treated area; they are made of materials specifically tested for modulable radiofrequency.

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### RF COMB TREATMENTS

The treatments with this technology provide for various alternate heating (through radiofrequency) and cooling (through Peltier cells) phases of the treated area without the need operator presence is constant; in fact, after positioning the plates on the area to be treated, it is enough to adjust the RF intensity, then, the device controls independently the emitted intensity and the cooling according to present parameters.

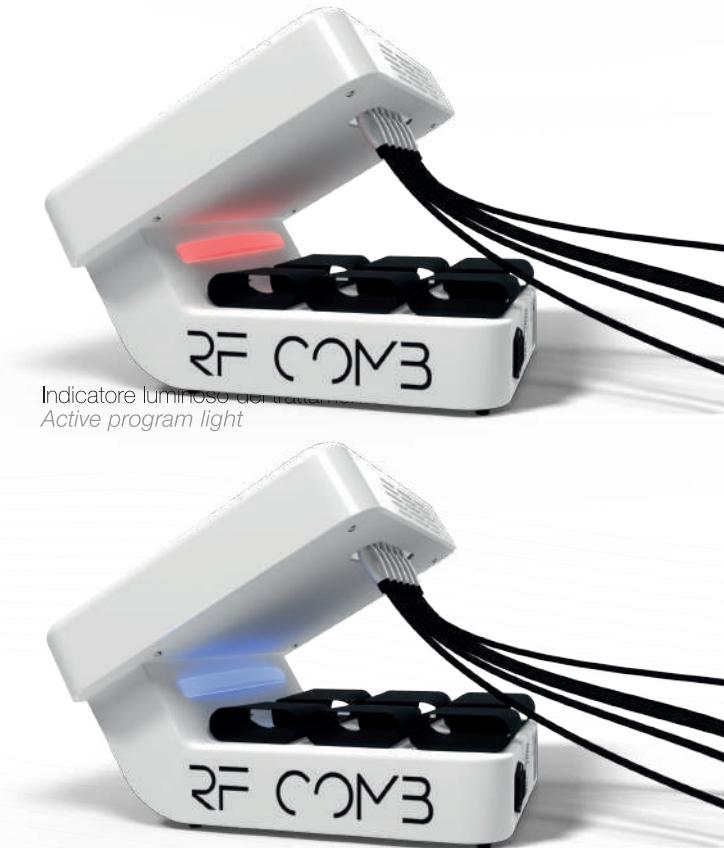
-**CELL**: for the treatment of the **cellulite imperfections**, it works with quick alternance of hot and cold; this reactivates the arterio-venous microcirculation, the venous-lymphatic drainage and pushes to a reorganisation of the cutaneous and suncutaneous structure. The treatment ends with a healing phase to eliminate toxins and metabolites.

-**LIPOLYSE**: program aimed to **localized fat excess**. It mainly focuses on local metabolic activity increase. The treatment starts with diluted cooling phase, that quickly increases the metabolism, and then restore a more correct blood supply.

-**SKIN TONE**: program aimed to **treatments of skin tone loss**. The targeted and deep radiofrequency allows a visible "shrink" effect and stimulates the new production of collagen fibres. The cooling phase thickens the interstitium and works on the tissue architecture and correct vascular bed.

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### General technical specifications

Input voltage:	100/240 V~
Main frequency:	47/63 Hz
Power consumption:	300 W
Electrical safety Class I	
Applied accessories type BF	
Continuous operating device	
Dimensions:	55X50X118